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

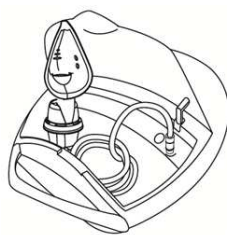
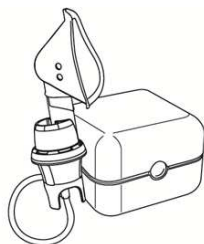
STANDARD ACCESSORIES SUPPLY

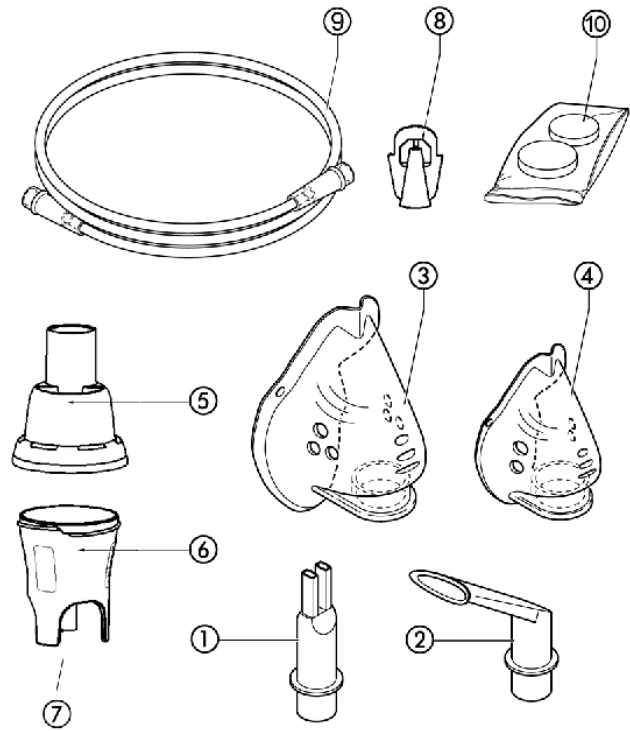- Pic.1 Nosepiece
- Pic.2 Mouthpiece
- Pic.3 Adult mask
- Pic.4 Pediatric mask
- Pic.5 Medicine Cup - topside
- Pic.6 Medicine Cup - underside
- Pic.7 Medicine Cup - air entrance
- Pic.8 Pisper
- Pic.9 Connecting Tube
- Pic.10 Air filter - Replacement

OPTIONAL ACCESSORIES

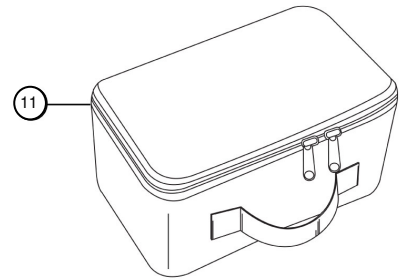- Pic.11 Bag

Warnings before use

Children and disabled persons should always use the unit under strict supervision of an adult who had read this manual.

The device is designed to be used by person with minimum age of 16 years old and with at least 8 years of study. Medicine cup and all accessories have been NOT sterilized. Before use read "Cleaning and maintenance" paragraph. Medicine cup and accessories can be reused after an appropriate cleaning. It is recommended a personal use.

The appliance is a Medical device of class IIa, which converts medicine from the fluid form to vapour in order to administer it directly into the breathing apparatus. The appliance nukes medicines both in suspension and in solution.

The therapeutic effect could be reduced with oily and/or pasty medicines.

The appliance is NOT a nebulizer for essential oil.

Don't nuke liquid and/or alcohol solutions.

The appliance generates compressed air. Secondary consequences could come for the intolerance to the medicine used.

Our policy is one of continuous improvement and therefore, the product inside may vary slightly from that shown.

Warnings to read before use

1. Carefully read the instructions before use. Keep this instruction manual for future references.
2. After removing any packaging materials, ensure that it has not been damaged. If in doubt, do not use it and contact a technician or an authorized service.
3. Use the device only as inhalator for therapeutic purposes. Medicine type, mode and time of application are only under medical limitation. Installation operations have to be done according to the instruction in this handbook and only for the uses declared from the constructor.
4. This device is unsuitable for use with inflammable anaesthetic mixtures containing air, oxygen or nitrogen protoxide.
5. The proper functioning of device can be affected by electromagnetic interference caused by malfunctioning of your television, radio, etc. If this happens, try to move the device until the interference disappears, or try to connect to a different outlet.
6. Never use extension cords or adapters. It is recommended to always carry the power cord to prevent dangerous overheating. Keep the cord away from hot surfaces.
7. Always disconnect the device from the main socket after use and before cleaning operations.
8. Do not touch the device if it accidentally falls into the water. Unplug it immediately and do not use afterward. Contact one of authorized service centre.
9. Do not use the device with wet hands.
10. Do not obstruct cooling grills.
11. During use, make sure that the unit is placed on a flat and stable surface to prevent any spillage.
12. Do not use the device when patient is sleeping.
13. Refer all servicing to the Service Centre. Do not open the equipment. If the device does not work properly turn it off and consult the instruction booklet (check additional sheet).
14. Do not leave any packaging materials (plastic bags, cardboard boxes, and so on..) within the reach of children to prevent dangers.
15. Keep accessories out of reach of children. Children and people with reduced physical or mental capacities need to use the device only under close surveillance of an adult who has read this instruction manual. Keep the Medicine cup out of the reach of children under 36 months because it contains small parts that could be ingested.
16. Always disconnect the device from the main socket when it is not used.
17. It is recommended to use one personal medication set of accessories.
18. Do not leave pharmaceutical residual into the medicine cup and/or into the accessories at the end of the therapy.
19. Cleaning and / or maintenance must be done only after turning off the device and unplug the power cord from the outlet.
20. Do not pull cable or appliance in order to remove the plug.
21. This is a medical device for home use and should be used on medical prescription. Must be run as indicated on this user's instructions. It's important that patients read and understand the information for the use and maintenance unit. Contact your dealer or Service Centre for any questions.
22. If the equipment's plug does not fit the socket, contact a qualified technician to substitute the plug. Do not use adaptors, multiple sockets and/or extensions.
23. Do not obstruct the air intake.

24. The installation must be done according to the instructions and use declared by the manufacturer. Improper installation and / or improper use may cause damage to persons, animals or things. The manufacturer can not be held responsible.
25. Check the instruction before using:
0/OFF Off switch **I/ON** On switch ~ AC
26. Do not use the device while bathing and showering
27. Do not expose the device to the action of dust (that could obstruct the cooling grids), heat (that could deform the housings), sunlight or other atmospheric agents that could compromise the safety of the device itself.
28. Do not use the device nearby sources of potential interference (example: heat sources like fireplaces, electric or gas heaters; steam sources like boilers or kettles, eletromagnetic radiation sources like microwave ovens, Wi-Fi devices).
29. Keep away from children, disabled persons.
30. Keep away from pets that could contaminate the surface in contact with the patient, close the ventilation grid, or in general avoid the right performance of the therapy.

Warnings

The device is not a lifesaver. It is recommended to keep another disposal or another medicine in case of impossibility to finish the therapy.

The therapy must be prescribed by a doctor and must indicate the mode and the time of the cure.

The device has been projected for domestic use. The maximum continuing operation is 20min. The minimum break after the operation is 40min.

Change of the number of the motor ride due to brownout etc, cannot change appliance performances.

Use the device against one or more of the previous warnings list could cause a bad function of the device , a wrong Aerosol therapy and injury to the patient.

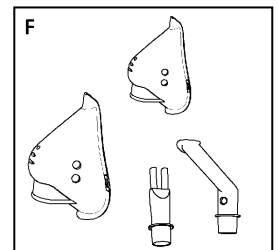
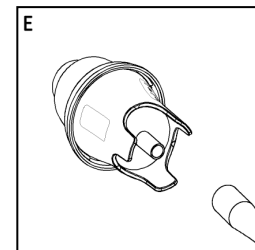
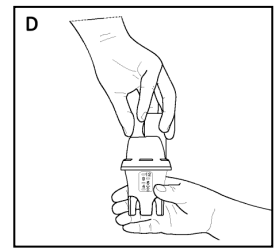
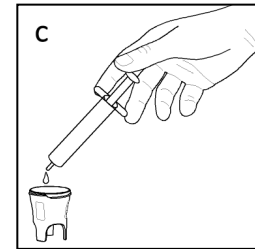
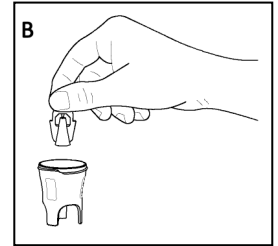
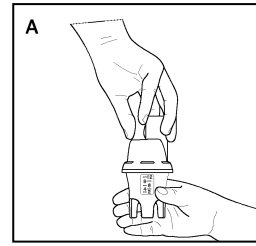
Warmly suggest the full respect of the warnings and the medical prescriptions to avoid serious problems.

Safety warnings:

- The product includes small parts that can be ingested or inhaled: keep product and accessories out of the reach of children.
- Power supply cord can be source of strangulation risk: keep under surveillance children and not self-sufficient people.

Use instructions

1. Put the device on a flat, stable surface.
2. Ensure that the rated voltage corresponds to the mains voltage supply.
3. Connect the device into the main socket.
4. Open the medicine cup unscrewing the two parts (A).
5. Insert the pisper in the nozzle of medicine cup – underside as illustrated in Figure (B)
6. Fill in the lower part of the medicine cup with the pharmaceutical (C) Note: The notches on the lower part of medicine cup are only indicative.
7. Close the medicine cup screwing back on the two parts (D)
8. Connect the air cable (9) to the medicine cup (E).
9. Connect the air cable (9) to the air exit of the appliance
10. Insert on the medicine cup the needed accessory: mask, mouthpiece or nosepiece (F).
11. Turn on the device moving the switch to ON position.
12. After the application, turn off the device and unplug. Before restarting the appliance make sure it has cooled to room temperature environment.

13. **Optional accessories:**

Some products are available in bag with for a more comfortable transportation.

ATTENTION:

Do not inhale horizontally.

Do not tilt the medicine cup over 45°.

Keep the accessories well ventilated, protected from heat and direct sunlight (F).

■ WARNING FOR A GOOD APPLICATION OF THE THERAPY

- Sit down in a comfortable and upright position.
- If a mask (adult or children) is used, be sure that it adheres to the face. Mask has holes for the air flow during the expiration phase. Breathe easy and have a short pause at the end of each breathe.
- If the mouth piece is used, keep it lightly among the teeth with tight-lipped. Respiration must be done through the mouth.
- If the nosepiece is used, put it softly on the nostrils. Don't insert it in the nose. Respiration must be done through the nose.
- Therapeutic application ends when the medicine came out in a discontinuous mode (spluttering).
- At the end of the application, carefully clean and disinfect the accessories as stated in the chapter "Cleaning and Maintenance"

■ CLEANING AND MAINTENANCE

After each therapy remove the device from the socket and clean the device and the accessories. **Attention:** use only the accessories supplied with the product, other accessories can lead to ineffective therapy.

CLEANING OF THE MOUTHPIECE AND NOSEPIECE

Use drinking water to carefully clean these accessories, when used. They can be sterilized by boiling for 5-10 minutes into the water or by immersion in a chemical disinfectant (oxidising electrolytic chloride) purchased from a pharmacy (e.g.: Amuchina®, Milton®), following product instructions and rinsing off with drinking water. Leave to dry naturally.

CLEANING OF THE MEDICINE CUP:

After use, open the three parts, wash with soap and hot water and rinse it carefully. Remove excess of water with a soft, clean and dry cloth and leave to dry naturally in a clean place. To sterilize, cover the medication container with a cold sterilizing solution, following the instructions on the sterilizer pack. It is recommended to use one personal medication container for each patient, to be use for 6 months or 120 treatments. Do not use boiling water to clean it.

CLEANING MASK AND HOSES:

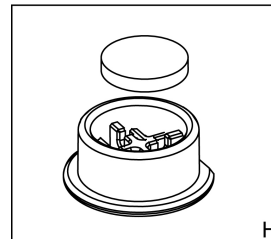
Carefully wash with disinfectant solution only.

CONTAMINATION BY MICROBES:

In case of pathologies of microbial or viral origin, where infection is possible, the device and its accessories should not be shared with other patients.

■ SERVICING

This device has an expected service life of about 2.500 therapy cycles, basing on an average therapy duration of 8 min. Parts as filter and accessories must be replaced periodically (see “CLEANING AND MANTEINANCE”).


FILTER REPLACEMENT PARTS (H)

The filter has to be substituted after 30 hours of use or when it becomes grey.

For filter replacing, raise the lid using a flat screwdriver as shown in the picture I.

Remove the filter and fit the new filter (10).

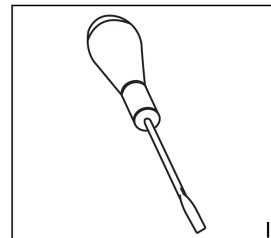Fit the filter carrier lid into its housing making sure it is adherent to the surface.

■ DEVICE’S TECHNICAL FEATURES

Piston rotary presser, without lubrication.

Medical device of class IIA according to Directive 93/42/CEE annex IX

Medicine Cup: Particles dimension < 5µm MMAD (EN13544-1)

Intermittent use: 20minutes - ON / 40minutes – OFF

Nominal voltage: 230 V ~

Frequency: 50 Hz

Power: 120 VA

Operating conditions:

Ambient temperature: min 5°C – max 40°C

Air humidity: min 15% RH – max 93% RH

Atmospheric pressure: min 700hPa – max 1060hPa (max operating altitude ≤ 2000m)

Storage conditions :

Ambient temperature: min -25°C / max +70°C

Air humidity: min 0% RH – max 93%RH

Atmospheric pressure: min 500 hPa – max 1060 hPa

■ SYMBOLS

	Alternate current		Medical CE mark ref. Dir 93/42 CEE
	Double insulation		Do not use the appliance when taking a bath or shower
	BF applied part		Switch ON
	Obligation to read instruction manual		Switch off
	General warning symbol		Read instruction manual
IP21	Protection against solid and liquid bodies penetration (protected against solid bodies over 12mm. protected against access with a finger; protected against vertically falling water drops.)		Keep dry
	This product (including its related accessories) must be delivered at the end of its service life at a collection point for the recycling by electric and electronic devices, by the user or by His dealer.		

■ PROBLEMS, CAUSES AND SOLUTIONS

The device is not turning on:

- Make sure the plug has been correctly inserted in the main supply
- Make sure switch is on the position turned on (I)

The device is turned on but it's not nebulising:

- Make sure to have inserted the pisper into the medicine cup (8)
- Make sure the air connection pipe is not bended or crushed (9)
- Make sure air suction filter is not stopped or dirty .In that case replace it with a new one (10)
- Make sure the medicine cup has been filled in with medicine.

Note: If the device shall not take again his correct functioning in spite of inspections effected, contact the After Sales Service.

■ MANUAL AND DECLARATIONS OF THE CONSTRUCTOR – ELECTROMAGNETICS EMISSIONS

The appliance for aerosol therapy can be use in the electromagnetic setting here indicated:

Emission test	Compliance	Manual to the electromagnetic setting
Irradiate/conducted emission	Group1	The disposal used the energy RF only for its internal function. So its RF emissions are very low and they do not cause any interferences to other electrical appliances.
Irradiate/conducted emissions CISPR11 Harmonic emissions IEC/EN 61000-3-2 voltage/flicker fluctuations IEC/EN 61000-3-3	Class (B) Class (A)	The appliance is adapt to be used in all the setting ,include the domestic environment and all the settings connecting to the distribution public net which give alimentation to the environment used for domestic use
Electrostatic discharges(ESD) IEC/EN 61000-4-2	±6kV in contact ±8kV in air	The floors should be done in wood, cement or ceramics. If the floors are cover by synthetic material, the relative humidity should be as a maximum of 30%
speed/burst transitors IEC/EN 61000-4-4	±2kV power	The supply should de typical of an hospital or a commercial setting
Surge IEC/EN 61000-4-5	±1kV differential mode	The supply should de typical of an hospital or a commercial setting
Voltage hole, short breaks and voltage change IEC/EN 61000-4-11	<5%Ut for 0.5 cycle 40%Ut for 05 cycle 70%Ut for 25 cycle <5%Ut for 5 sec	The supply should de typical of an hospital or a commercial setting If the user need a continuing working of the device, it's recommended to use the appliance under a continuity group.
Magnetic field IEC/EN 61000-4-8	3A/m	The magnetic field should be typical of an hospital or an commercial setting.
Conducts immunity IEC/EN 61000-4-6	3Vrms 150kHz to 80 MHz 8 (for appliances life-supporting)	
Irradiate immunity IEC/EN 61000-4-3	3V/m 80 MHz to 2.5GHz (for appliances NOT life-equipment)	

Note UT is the value of the supply voltage

Elettromedical devices require special care. During installation and use with respect to EMC requirements, it therefore required that they are installed and/or used according to the manufacturer's specification. Potential risk of electromagnetic interference with other devices, in particular with other devices for analysis and treatment. Radio and mobile telecommunications devices or portable RF (mobile phones or wireless connections) may interfere with the operation of elettromedical device.

Manufactured by: EP S.p.A. via Del Commercio 1 Travagliato (BS) ITALY
Tel. 030/2583990 - Fax. 030/2584012 - Website: www.emeditaly.com

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Emed is a registered trademark of EP S.P.A.